



The Care and Maintenance of Liko Slings

English
7EN160132-02
2007-09-27

Check all slings regularly, particularly after laundering.

Important checkpoints:

- Fabric
- Straps
- Seams
- Loops

Check all points closely for wear and damage.

Laundering:

Follow the laundry instructions on the sling.


Laundering temperature: usually 60°-80°C (140°-176°F).

Do not use bleach.

Tumble-drying: usually low temperature.

Drying cabinet may be used.

Padded and reinforced parts may be dried by rolling them in a towel and pressing out moisture.

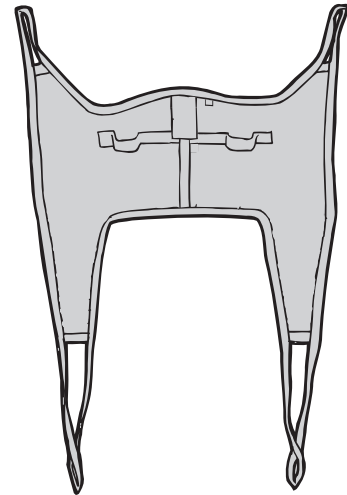

General Advice:

Slings should be considered as consumable products that are of critical importance to patient safety.

Slings made of nylon and polyester fabric have a longer life span than those made of cotton fabric.

The handles found on some slings are intended to be used to steer, not lift, patients. Excessive weight on the handles may break the seams.

For safety and hygiene reasons, individual slings should be used for each patient.

For further information on the care and maintenance of Liko slings, please contact Liko or your local Liko representative.



www.liko.com

Manufacturer:

Liko AB
975 92 Luleå
Sweden
info@liko.se

National Distributor in USA and Canada:

Liko North America
122 Grove Street
Franklin, Ma 02038, USA
Phone: 508-553-3993, 1-888-545-6671
Fax: 508-528-6642
info@liko.com

Liko Distributor in UK:

Liko (UK) Ltd.
Brunel Centre,
Stroudwater Business Park
Stonehouse, Gloucester
GL10 3RU, United Kingdom
Phone: 01453 827 272
Fax: 01453 828 844
info@liko.co.uk